




Echo® MiniTech Pro BT™ Instructions

Bluetooth Receiver



Product Code: 754

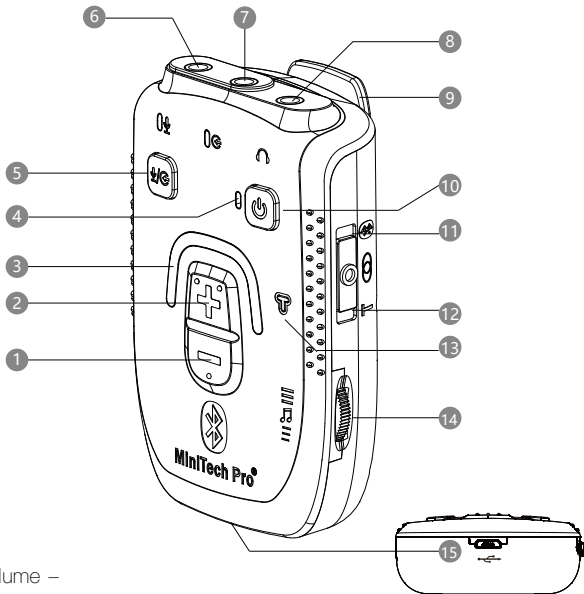

- | | | | |
|---|---|----|--|
| 1 | Volume - | 10 | Power On/Off |
| 2 | Volume + | 11 | Push Up to Pair with Bluetooth |
| 3 | Volume Indicator
(Blue Lights-Green Lights-Red Lights) | 12 | Push Down for "T" Coil |
| 4 | Power On/Off LED Indicator | 13 | Bluetooth Mode and "T" coil
Mode LED Indication |
| 5 | Switch between Microphone & Line in | 14 | Tone Control |
| 6 | Microphone in | 15 | USB Charging Jack |
| 7 | Line in | | |
| 8 | Headset/Neckloop | | |
| 9 | Belt Clip | | |

Power On

Press the power 10 button about 5 seconds until the Power On/Off lights 4 green

Charging Your MiniTech Pro BT

Connect the USB charging cable to charging USB socket 15
The Power On/Off light 4 stays red while charging. The red light will turn off once it is fully charged.

Noted: The Power On/Off light 4 keeps red when the battery is low.

Pairing Your MiniTech Pro BT

- 1) Press the power button to On
- 2) Push up and hold for 4 seconds to pair with Bluetooth 11 until the connection LED Indication 13 flashes blue quickly.
- 3) Open the Bluetooth devices within range and select “MiniTech Pro BT” from the list of devices shown.
- 4) When the pairing is completed, the LED Indication 13 will flash blue twice every 3 seconds.

Switch On/ Off Bluetooth

- 1) Press the Power button to On
- 2) Push up and hold for 2 seconds to turn the Bluetooth On 11 until the connection LED indication 13 flashes blue twice every 3 seconds.
- 3) Push up and hold for 2 seconds to turn the Bluetooth Off again 11 until the connection LED indication 13 goes off.

T Switch 12

When push Down for “T” Coil, the LED Indication 13 changes to red.

When T-coil, microphone or line input activated then Bluetooth mode will switch off automatically.

Clear the pairing list:

Push up and hold the pairing switch for 10 seconds until the blue LED starts flashing slowly.

Answering and ending calls:

Push up the pairing switch, for 1 second then let go.

Hearing Products International Limited declares that the Echo[®] MiniTech Pro BT[™] complies with all relevant EU directives.

The full EU declaration of conformance for the Echo[®] MiniTech Pro BT[™] is available from Hearing Products International Ltd.

Hearing Products International Limited

Echo House, 26/27 Haigh Park, Stockport, SK4 1QR, England

Tel. +44 (0) 161 480 8003

Fax +44 (0) 161 480 8006

www.hear4you.com



Hearing Products International Ltd. reserve sole right to alter prices, design and/ or technical specifications or colours without prior notice. Echo[®]MiniTech Pro BT[™] Instructions © Copyright Hearing Products International Limited. E&OE 07/01/19. Version 1. UK: Waste electrical products and batteries should not be disposed of with household waste. Separate disposal facilities exist, for your nearest facilities see www.recycle-more.co.uk or Hearing Products International Limited for full details www.hear4you.com.

Printed in China