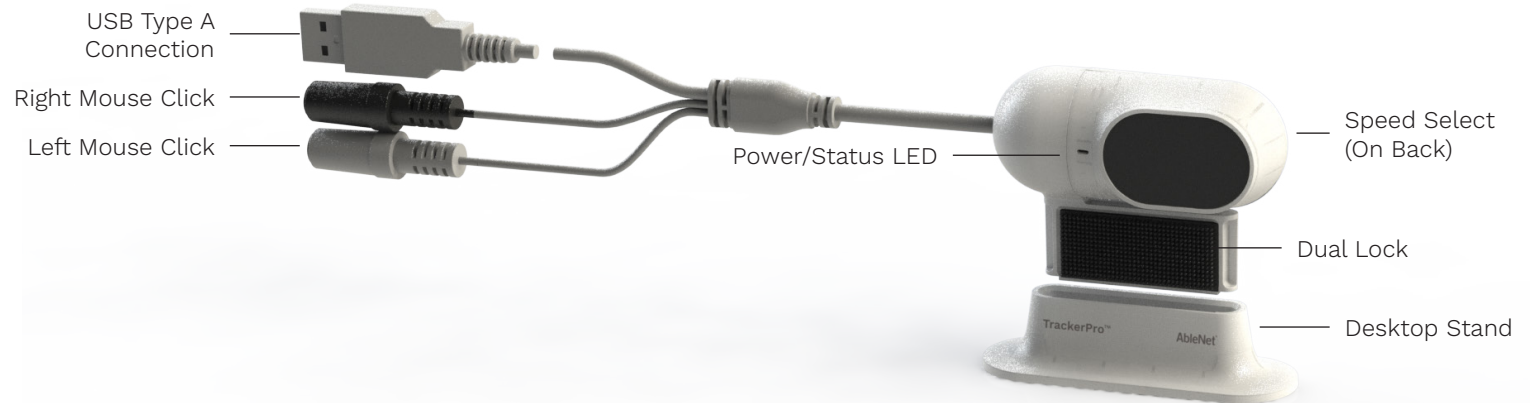
AbleNet[®]

TrackerPro™ 2

www.ablenetinc.com
(800) 322-0956

Getting Started:

1. Place TrackerPro 2 directly in front of the user. TrackerPro 2 can be attached to a monitor using the included dual lock or placed on a flat surface using the included desktop stand.
2. Plug TrackerPro 2 into available USB Type A port on computer. The Power/Status LED on TrackerPro 2 will turn red.
3. Place a TrackerPro 2 reflective dot on forehead of user.
4. Position the mouse cursor in the center of the device screen before the user starts.
5. Position user 12-in to 36-in (30-cm to 91-cm) away from TrackerPro 2.
6. The Power/Status LED on TrackerPro 2 will turn green once it detects the reflective dot on the user.
7. Small head movements will move the mouse cursor on the computer. If the user is unable to move the mouse cursor across the entire screen, try adjusting the mouse cursor speed.

Cursor Positioning:

When a user first starts or after using TrackerPro 2 for a

period of time, the user may find that the mouse cursor does not reach the far edges of the device screen. To re-orient the mouse cursor:

1. The user should move their head as far up and then as far down as it can go. The mouse cursor may stop and push up against the edge of the screen on the up or down movement of the head. Repeat the up and down motion multiple times. This movement will re-orient the mouse cursor in the vertical direction.
2. The user should move their head as far left and then as far right as it can go. The mouse cursor may stop and push up against the edge of the screen on the left or right movement of the head. Repeat the left and right movement multiple times. This movement will re-orient the mouse cursor in the horizontal direction.
3. If neither of the above steps works to reach the edges of the screen with the mouse cursor:
 - a. On the back of TrackerPro 2, turn the small toggle switch to I
 - b. On the device, go into the setting for the mouse tracking speed and turn it up

Mouse Click Options:

- Plug one or two switches into TrackerPro 2. The white switch jack is for left mouse click and the black switch jack is for right mouse click.
- For use without switches, use a dwell selection software.
 - Windows: Dwell Clicker 2 (purchased separate)
 - macOS Mojave or newer: Dwell selection is part of the accessibility settings for the keyboard
 - Google Chrome OS: Dwell selection is part of the accessibility settings

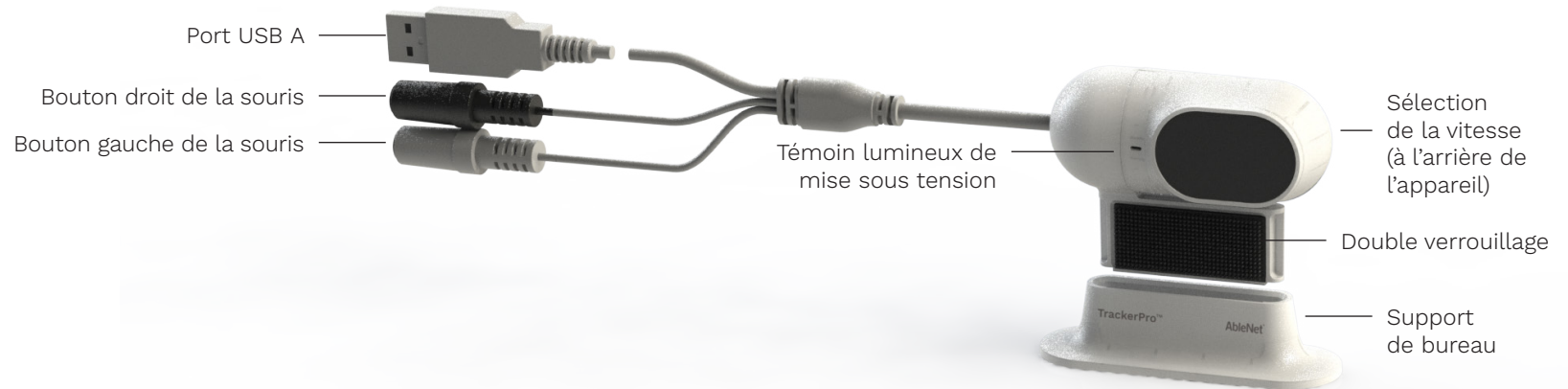
Mouse Cursor Speed Adjustment:

- On the back of TrackerPro 2, move the speed selector switch to change mouse cursor speed.
 - I = fast
 - II = slow
- For more control, find the mouse cursor speed adjustment within the computer operating system

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(800) 322-0956
(651) 294-2200
www.ablenetinc.com

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FCC Title 47 Part 15c

Instructions for Use Information

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

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For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment (EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union (EU). If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

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